Univox[®] Safewake Univox[®] Safewator and vibrator and wakening device alarm indication Wakening moke alarm indication

UniVox[®] SafeWake is a practical device for hearing impaired who needs help to wake up - in hotels or at home. SafeWake works in the same way as an ordinary wakening device with vibrator but has one additional feature: it also works as an indicator for smoke alarms.

As the built-in microphone detects the sound, either from the alarm of the clock itself or from the smoke alarm, the electronics inside the clock activates both acoustical alarm and vibrator. When SafeWake operates like an ordinary wakening device, the vibrations stop as the on/off button is pressed down. If however the clock reacts on the smoke alarm, the vibrator keeps on vibrating until the signal from the smoke alarm stops.

The SafeWake set consists of clock with hard wired vibrator, power supply and user instruction delivered in a practical carrying case.

Features

- Clock with acoustic alarm and hard wired vibrator for wake up and indication of smoke alarm
- Built-in microphone on the rear side of the clock
- Built-in chargeable backup battery (operates clock and microphone for min. 12 hours and vibrator for min. five minutes)
- Vibrator only reacts on acoustical input (for wake up as well as for smoke alarm)
- The electronics reacts on the clocks own alarm signal as well as on acoustical smoke alarms which are approved (see below)
- The vibrator reacts at >70dB SPL (Sound Pressure Level) which equals normal speach from approx. 1 metre distance (complying with demands from the Swedish Fire Safety Authorities and The Swedish National Board of Housing, Building and Planning)
- Frequency response for built-in microphone >2kHz
- Vibrator reacts on continous sound for more than 2 seconds
- Wake up function and smoke alarm indication works separately - smoke alarm indication works even if the on/off button is pressed down and the wakening function is deactivated.

For additional information, please refer to user guide/installation guide and CE certificate which can be downloaded from "Product databank" at <u>www.edin.se</u>. If required, spare part list or other technical documents can be ordered through support@edin.se.



Mail address Stockby Hantverksby 3 SE-181 75 LIDINGÖ Sweden

Visiting address Förrådsvägen 2 B SE-181 41 LIDINGÖ Sweden

Phone +46 8 7671818 Fax +46 8 7671820 E-mail info@edin.se Internet www.edin.se



UniVox[®] SafeWake

System Overview

UniVox® SafeWake



Part no.Description345001SafeWake complete set

Part no. Description

345010	Alarm clock with hard wired vibrator (vibrator cable length: 2m)
389007	AC Adaptor, 230V/4,7V DC 200mA (cable length: 2m)
345020	Carrying case with shoulder strap
902441	User instruction leaflet (english)

CE

Spare parts



