



# UniVox® DLS-30

## Complete loop amplifier kit with built-in automatic scart control

UniVox® DLS-30 is a complete assistive listening device for hearing aid users watching TV.

DLS-30 has a built-in automatic scart (Euro connector) control which is easily connected to the scart output of your TV. The autoscart function is needed whenever you have one or more signal sources (digibox, VHS etc.) connected to the TV and secures the right sound in the loop. If you have several signal sources, make sure they are all serial connected. All but the last unit must have double scart connectors (see ill. page 2). In the case a serial connection is not possible, a microphone (accessory) can be connected to the loop amplifier and attached to your TV's speaker grid.

Place the loop pad in your favourite chair and tune your hearing aid into T-position. For coverage of a larger area a sofa loop (accessory) can be connected to the amplifier instead of the loop pad. Thanks to the built-in level control (dual action AGC) DLS-30 delivers a very smooth and pure sound.

If you have a LCD/plasma screen TV and have sound delay problems, we recommend UniVox® DLS-30TV with the same characteristics as DLS-30, but with the possibility to synchronize the sound.

### Features

- Complete induction loop kit with loop pad for TV/digibox/satellite/VHS/DVD
- Automatic scart control built-in (cable included)
- 2 inputs for microphone/stereo/MP3 etc.
- Automatic level control (dual action AGC)
- LED's for indication of mains power connection and loop current
- Volume thumbwheel (digits for easy setting of right level)
- Loop pad can be replaced by sofa loop cable (length 8m + extension cable 10m)

For additional information, please refer to user guide/installation guide and CE certificate which can be downloaded from "Product databank" at [www.edin.se](http://www.edin.se). If required, spare part list or other technical documents can be ordered through [support@edin.se](mailto:support@edin.se).



**Mail address**  
Stockby Hantverksby 3  
SE-181 75 LIDINGÖ  
Sweden

**Visiting address**  
Förrådsvägen 2 B  
SE-181 41 LIDINGÖ  
Sweden

**Phone** +46 8 7671818  
**Fax** +46 8 7671820  
**E-mail** [info@edin.se](mailto:info@edin.se)  
**Internet** [www.edin.se](http://www.edin.se)



## Product overview

### UniVox® DLS-30



DLS-30 loop amplifier



DLS-30, rear side



Loop pad



Sofa loop cable

Part no	Description
203010	UniVox® DLS-30 loop amplifier with built-in autoscant, two microphone/line inputs, power supply and loop pad

#### DLS-30 loop amplifier

Size:	145x60x30mm (WxDxH)
Weight:	180g excl. power supply
Scart cable length:	50cm

#### Loop pad

Size:	400x400x5mm (WxDxH)
Cable length:	10m

203010 S	Same specification as part no 203010 but with sofa loop
----------	---

#### Sofa loop

Cable length:	8m + 10m extension cable
---------------	--------------------------

203010 R	Same specification as part no 203010 but with room loop
----------	---

#### Room loop

Cable length:	30m
---------------	-----

Versions for use in the UK, USA and Australia are available.


### Alternative for LCD/plasma TV with UniVox® DLS-30TV

Part no	Description
203040	UniVox® DLS-30TV loop amplifier with digital sound synchronization, built-in autoscant, two microphone/line inputs, power supply and loop pad
203040 S	UniVox® DLS-30TV loop amplifier with digital sound synchronization, built-in autoscant, two microphone/line inputs, power supply and sofa loop
203040 R	UniVox® DLS-30TV loop amplifier with digital sound synchronization, built-in autoscant, two microphone/line inputs, power supply and room loop

## Serial connection of TV / Satellite / Digital box / VHS / DVD

Ensure that the TV works correctly together with your Satellite receiver/Digital box/VHS/DVD before you start the installation of DLS-30. The sound sources that are connected to the TV have to be serial connected (see illustration below). If there are more than one scart output on your TV, always use scart output no 1. All signal sources but the last have to have two scart connectors each (in/out).

If the connections are correct the picture and sound change automatically according to the chosen signal source. For example if you start the VHS, sound and picture in your TV change automatically to the sound and picture of the VHS movie.

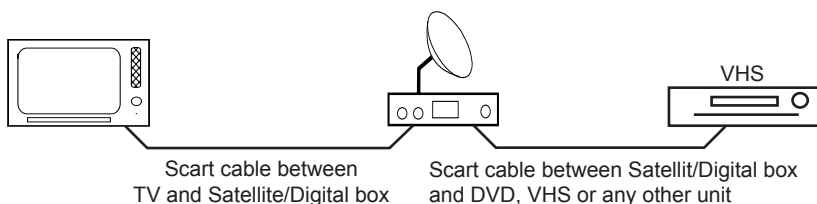

### Accessories

Part no	Description
283002	Loop pad
283005	Loop pad cover
283220	Sofa loop incl. extension cable, 8+10m
283230	Room loop, 2x0,75mm <sup>2</sup> , cable connector 2-pin DIN, grey, 30m
242401	13A, electret microphone with welcro adhesive, cable 3m
581050	Electret tie-clip microphone with clip, cable 1,5m
241501	13T, electret boundary microphone, cable 3m
281005	Extension cable for microphone, cable 3m
281006	Extension cable for microphone, cable 6m
281015	Audio cable for connection to stereo, 3,5mm to 2xRCA, cable 1,5m

